



How to Import an Implant Library for Exocad and 3Shape













How to import the Creomc Library into exocad

1. Insert the included USB Drive to your computer. The Library in the usb will contain the following files:

 01_3SHAPE	7/6/2015 8:57 AM	File folder
 02_EXOCAD_Z Axis	7/21/2015 4:41 PM	File folder

2. Open the 02_EXOCAD_Z_Axis folder you will find these files:

 CREOMC_103_DT	7/21/2015 1:13 PM	File folder
 CREOMC_108_OT	7/21/2015 1:18 PM	File folder
 CREOMC_201_AS	7/21/2015 1:27 PM	File folder
 CREOMC_205_3I	7/21/2015 1:32 PM	File folder
 CREOMC_209_NB	7/21/2015 1:36 PM	File folder
 CREOMC_210_ACT	7/21/2015 1:39 PM	File folder
 CREOMC_212_ST	7/21/2015 1:43 PM	File folder
 CREOMC_213_SS	7/21/2015 1:54 PM	File folder
 CREOMC_214_Z	7/21/2015 1:59 PM	File folder
 CREOMC_219_B	7/21/2015 2:03 PM	File folder

How to import the Creomc Library into exocad


3. Open Local Hard Drive (C:) then go to Program Files (x86)



4. Open the folder that says DentalCad and select the library folder



5. Select implant folder, then copy and paste all of the exocad library folders from our enclosed USB (Step 2).

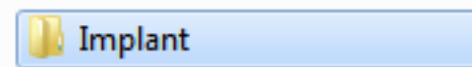

How to import the Creomc Library into 3Shape

1. Open The Dental Systems Control Panel Manager then scroll to tools and select import/export



How to import the Creomc Library into 3Shape

1. On the next page, click Import materials and import the .dme file located in the 01_3SHAPE folder on the given USB.

Import materials

To import new materials, click on "Import materials" and browse the desired file (extension ".dme"). Then select the materials from the list that appeared on a screen and click "Import". Typically it is recommended to select all. You can also download the materials directly from FTP - go to Site Settings -> Sites page and click "Download Materials".



Import materials

01_3SHAPE

CREOMC_07032015.dme


How to import the Creomc Library into 3Shape

1. Press import on the popup box. On the next popup, check the box that says “Do the same for all similar cases” and click on Keep name.

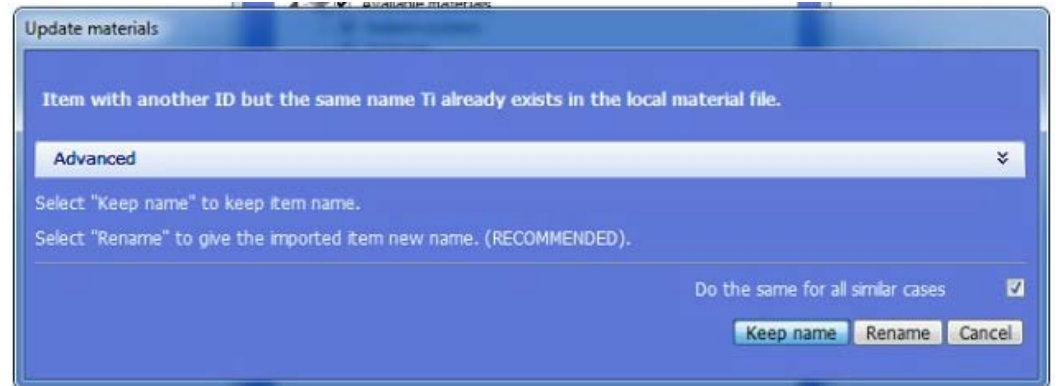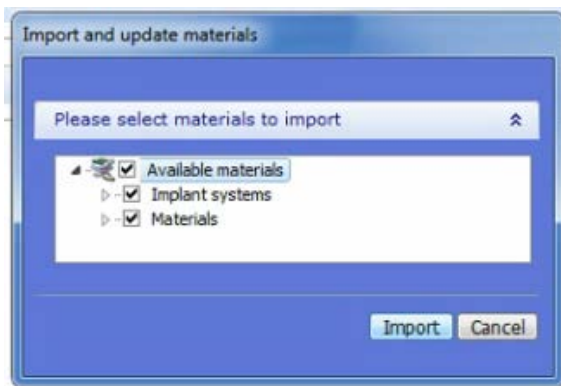


How to import the Creomc Library into 3Shape

1. Press ok and yes on the next boxes.

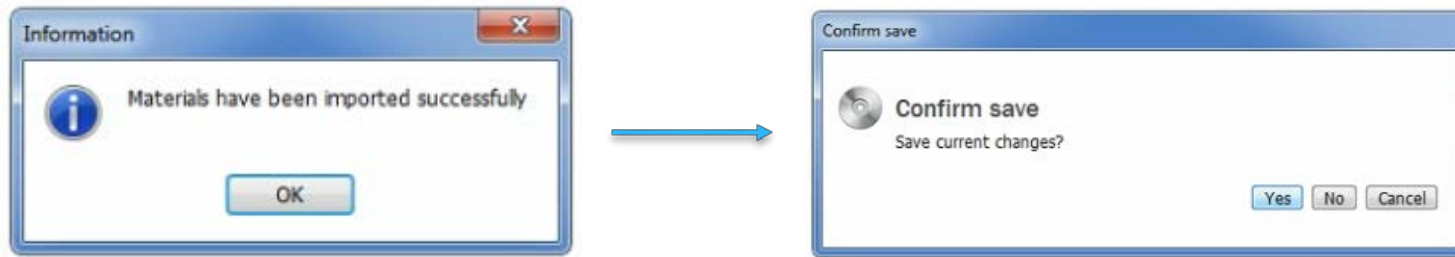

1. Go to Abutments then Implant systems and scroll to the left to check to see if the library is imported.

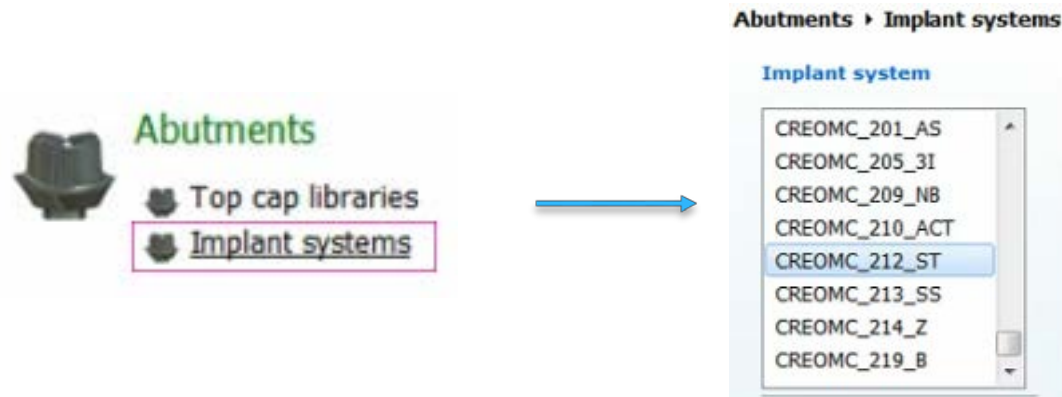

Thanks for Watching

Call us at 212-302-3860 if you have any questions

info@creomc.com